

Unofficial translation of the German package leaflet
INSTRUCTIONS FOR USE - Please read carefully.



MediGel®

RAPID HEALING

Hydroactive lipogel with zinc and iron for the treatment of acute and chronic wounds

Intended use

MediGel® Rapid Healing is a lipogel for the topical treatment of acute wounds (e.g., grazes, cuts, scratches and bites*, as well as lacerations and tears), first- and second-degree burns (with blistering), and sunburn. MediGel® Rapid Healing can also be used for the treatment of cleaned, uninfected chronic wounds with a moderate amount of weeping (grade 2 pressure ulcers); treatment of chronic wounds must only take place under the supervision of medical professionals.

* For bites, it is necessary to see a doctor for initial treatment, as there is a high risk of infection.

Properties

MediGel® Rapid Healing is a white odourless lipogel with hydroactive properties that combine the moisturising properties of a gel with the nourishing properties of a cream. It is suitable for use in both dry wounds and wounds with a moderate level of weeping. MediGel® Rapid Healing uses many of the principles of moist wound healing for optimal healing of the injured skin. The low pH value, supported by the presence of iron and zinc ions, activates the natural healing process and triggers the formation of new skin. Natural nourishing oils and vitamin E support the regeneration of the damaged skin. MediGel® Rapid Healing is tolerated very well by all skin types

and can therefore be used by the whole family, including children and those with sensitive skin.

Thanks to the physical properties of the cream, MediGel® Rapid Healing creates the following conditions for rapid wound healing:

- Rapid pain relief and cooling sensation
- Restoration of a suitable microclimate in the wound
- Promotes wound healing
- Reduces swelling
- Reduces the risk of infection
- Reduces scarring

Instructions for use

Wash your hands before and after treating a wound. Before treatment with MediGel® Rapid Healing, the wound should be thoroughly cleaned. Apply approximately a 2 mm-thick layer of MediGel® Rapid Healing to the cleaned wound, making sure to go beyond the edges of the wound (the wound heals from the outside in). In the first days following the injury, MediGel® Rapid Healing can be applied several times a day.

The product can be used in all stages of wound healing until the injury has healed completely. Make sure that the contents of the pack is removed hygienically, avoiding any contact between the tube opening and the wound.


It is advisable to use MediGel® Rapid Healing together with a standard plaster or dressing in order to provide additional protection for the wound.

Uninfected chronic wounds with a moderate amount of weeping (grade 2 pressure ulcers): treatment must only take place under medical supervision.

MediGel® Rapid Healing contains no skin irritants and can be used on mucous tissues (e.g., in the nose) and on the mouth.




Clean

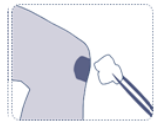

Dry



Apply

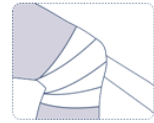

Cover

Precautions for use

- MediGel® Rapid Healing is intended for external use only and should not be ingested.
- Do not use in the event of known hypersensitivity to any of the ingredients. In the event of an allergic reaction (rash), stop using the lipogel. Carefully wash the lipogel off using water, and consult a doctor.
- Do not apply MediGel® Rapid Healing to the eyelids or the eyes. If MediGel® Rapid Healing comes into contact with the eyes, rinse the eyes with running water and consult a doctor.
- Do not combine MediGel® Rapid Healing with products containing silver, and do not use with occlusive dressings.
- For very large wounds, or wounds that are weeping heavily or are infected, you must consult a doctor before the use of MediGel® Rapid Healing, as additional treatment may be required in some cases.
- If the wound becomes red or swollen, or begins to ache, or if the wound does not begin to heal after a few days, you should consult a doctor.
- If you are pregnant or breast-feeding, you should only use MediGel® Rapid Healing after consulting your doctor.

Interactions

The product should not be combined with other creams or ointments.

Side effects

Possible allergic reactions; see the section "Precautions for use". If you experience any side effects, please inform your pharmacist or the manufacturer.

Storage instructions

- Protect against frost and heat. Store the lipogel in a dry place and in the original packaging. Do not store below 5°C or above 30°C.
- Close the tube after use. Only use the original lid to close the tube.
- Do not use MediGel® Rapid Healing after the expiry date on the pack.
- Shelf life of the unopened pack:
3 years after manufacture (tube)
2 years after manufacture (sachet)
- Shelf life after opening: 90 days after opening (tube)
For single use (sachet)
- Do not use if the tube is damaged. Make sure that the seal is not damaged.
- MediGel® Rapid Healing should be kept out of reach of children.

Ingredients









Trace element solution (distilled water, zinc, iron), octyldodecanol, hydrogenated palm glycerides, PEG-200 hydrogenated glyceryl palmitate, cetyl alcohol, hydroxypropyl starch phosphate, tocopherol, benzyl alcohol, methylchloroisothiazolinone, polyhexanide, silica

Pack sizes

Tubes of 5 g, 10 g, 20 g and 50 g, and 2 g sachet. Not all pack sizes may be marketed.

Last revised in: May 2017

Explanation of the symbols on the carton

-  Please follow the instructions for use
-  Expiry date
-  Batch code
-  Medical device
-  Do not use if the tube is damaged
-  Temperature limit 25°C
-  Manufacturer
-  For single use only (2 g sachet)

Manufacturer

MEDICE Arzneimittel

Pütter GmbH & Co. KG

Kuhloweg 37

D-58638 Iserlohn

Tel.: +49 (0) 2371 937-0

Fax: +49 (0) 2371 937-329

E-mail: info@medice.de

www.medice.de